


# Basic Customization Instructions for Snap + Core First



## Getting Started (User Initial Set Up)

- Follow the steps from the start-up page by selecting the arrow.
- You can use the pull-down menu to choose a language.

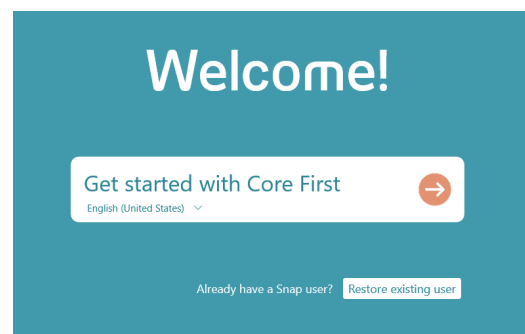

## Communication Pageset Overview

(Note: Pageset may vary with the grid size you choose.)

The screenshot shows the Snap + Core First Full application interface. The top bar includes navigation icons (back, home, search) and the title 'Core Words'. Below the top bar is a white message window with a 'Speak' button. The main area is a grid of communication tools and words. A 'Communication Tool Navigation Bar' is on the left, allowing users to scroll up or down. A 'Sync to back up your user pageset' button is at the top right. A 'Panel to modify or customize your pageset' is also visible. The grid contains various icons and words, including 'I', 'is', 'want', 'not', 'All Word Lists', 'you', 'can', 'like', 'more', 'Personal', 'it', 'do', 'go', 'a', 'Little Words', 'on', 'if', 'but', 'Description', 'he', 'have', 'stop', 'and', 'Word Forms', 'People', 'help', 'Actions', 'the', and 'Keyboard'.

Speaks the message in the white Message Window

Communication Tool Navigation Bar (scroll up or down)


Sync to back up your user pageset

Opens the panel to modify or customize your pageset

Page area where communication tool vocabulary and messages appear when selected from the toolbar (scroll up or down)

Recorded webinars are a great way to learn more about the communication tools, organization or customization ideas.

## Overview of the Modify Menu Panel

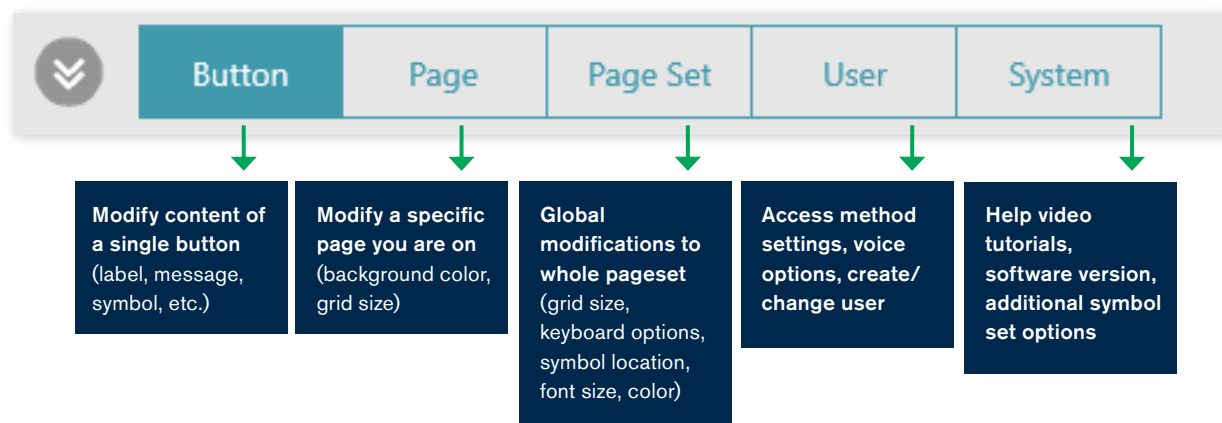

### Editing a Button in Snap + Core First

- Select the **Modify button** in the top right corner.
- Select the button you want to change.
- Type the new word (or phrase) beside Label – this is what is printed on the button.
- The button will “speak” the label unless you choose to change the Message.
- If you want to choose a different symbol (picture), select the symbol or you can take a picture using the camera.
- Select **Done** in the top right-hand corner.

### Searching for Vocabulary in Snap + Core First

- In the top left-hand corner, select the **Magnifying Glass/Search Tool**.
- Type in the word and select the word or message with the word from the drop down menu.
- Follow the path by selecting the button the system highlights with a blinking border.

### Changing the Grid Size of the Pageset

- Select the **Modify button** in the top right-hand corner.
- Select the **Page Set** tab at the bottom.
- Select **Grid Size** on the left.
- Explore the different grid sizes from 1x1 to 8x10.
- You can easily change the grid size anytime, without losing changes you’ve made to buttons.
- Select **Done**.

### Create a New User

- After the initial setup, if you want to create a new user, touch the **Modify button** in top right corner.
- Select the **User** tab at the bottom.
- Select **New User**.
- Follow the steps in the User setup Wizard to create a new user.